

CAVEX IMPRESAFE

Effective spectrum:

Bactericidal | Yeasticidal | Virucidal*
Complies with EN 13624, EN 13727, EN 14476, EN 14561,
EN 14562, EN 17111

* Enveloped viruses (incl. HIV/HCV/HBV)

Exposure time: 3% solution in 3 minutes
Composition in 100 g: 30.0 g Benzalkonium chloride,
auxiliaries and water

DANGER

- H302** Harmful if swallowed.
- H314** Causes severe skin burns and eye damage.
- H410** Very toxic to aquatic life with long lasting effects.
- P280** Wear protective gloves/protective clothing/eye protection/face protection.
- P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell.
- P301+P330 +P331** IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
- P303+P361 +P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.
- P305+P351 +P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P501** Dispose of contents/container in accordance with local and national regulations.

CAVEX IMPRESAFE



UFI: KYYW-28MC-100F-TSC7



United Disinfectant Manufacturers AG
Allmendstrasse 21 8320 Fehraltorf
Switzerland +41 (0)55 511 03 33
info@udm.swiss www.udm.swiss



United Disinfectant Manufacturers AG
Dr. Grass-Strasse 12 9490 Vaduz
Principality of Liechtenstein
info@udm.li www.udm.li

2460



cavex.nl



Manufactured exclusively for
Cavex Holland BV
Fustweg 5 2031 CJ Haarlem
The Netherlands +31 23 530 77 00
dental@cavex.nl cavex.nl

1 liter

CAVEX
HA0501.10



AXMKFP

Instructions



Squeeze bottle to dispense 30 ml.




Pour 970 ml of water (always first) in reservoir.




Pour disinfectant into reservoir.



Place impression in basket. Lower the basket with the impression into the fluid.

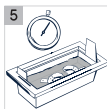

Leave impression for 3 minutes in the fluid for complete disinfection.

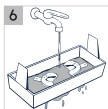

Take impression out of fluid and rinse thoroughly under running water of at least drinking quality for 10 seconds.



Shake off excess water and put impression in a plastic bag. Seal the bag airtight and mark the bag as "disinfected".

DOSING TABLE

		+		
3%	30 ml		970 ml	3 min



FOR PROFESSIONAL USE ONLY.

Not for use on invasive medical devices. Do not mix with other products. Make sure that all visible soiling has been removed before disinfection.



Store in cool, dry and well-ventilated facility. Keep out from direct sunlight, heat, and open flame. Storage temperature: < 30°C

A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use.