



HOW DOES WHITENING WORK?

In order to whiten teeth, the coloured molecules which have built up behind the enamel need to be dealt with. This can be done using hydrogen peroxide, which is included in the whitening gel. A reaction is triggered when you hold the whitening tray full of gel in your mouth. Energy is released during this process, and it is this energy that degrades the coloured molecules behind the enamel so that the teeth become whiter.



before

after

RESPONSIBLE WHITENING STARTS WITH YOUR DENTIST

Many whitening materials are available from a wide variety of sources, but it is only from your dentist that you will have the assurance of obtaining safe and legally approved products for responsible whitening. You can then be confident of achieving a lasting result and beautiful, naturally white teeth. Your dentist will also be able to inform you properly and guide you throughout the whitening treatment.

Your dentist will provide you with the Cavex Bite&White ABC Masterkit you need for your home whitening procedure. This kit consists of a tailor-made whitening tray, plus whitening gel containing hydrogen peroxide. Your dentist will talk you through the instructions, as well as providing practical advice for your home whitening sessions. After that, you can decide for yourself the time, location and length of your whitening sessions.

CAVEX BITE&WHITE. THIS IS WHY.

- Quick, safe and effective whitening
- Whitening under the guidance of your dentist
- Whitening wherever and whenever you like, with lasting results
- A complete 3-step whitening system featuring pre-care and post-care products
- Recommended by dentists all over the world for many years

PREMIUM WHITENING

YOUR IMPRESSION IS OUR SPECIALTY

PREMIUM WHITENING

HOW WOULD YOU LIKE TO HAVE BEAUTIFUL, NATURALLY WHITE TEETH?

WHITENING YOUR TEETH SAFELY

Wherever and whenever you like, guided by your dentist.



ZB954EN.04

Fustweg 5 | 2031 CJ Haarlem | Nederland | cavex.nl

YOUR IMPRESSION IS OUR SPECIALTY

CAVEX

NATURALLY WHITE TEETH, QUICKLY AND SAFELY THANKS TO THE CAVEX BITE&WHITE ABC MASTERKIT

BE SURE OF A BRIGHT SMILE WITH
NATURALLY WHITE TEETH

Smiling is healthy; it makes you happy and boosts self-confidence. That's why all of us are eager to have a wonderful smile. Beautiful, naturally white teeth are the basis of such a smile, and this can now be realised easily thanks to the Cavex Bite&White ABC Masterkit. Using this unique 3-step system, you can whiten your teeth wherever and whenever you like under the guidance of your dentist. In just a short time you will be enjoying a beautiful smile, with wonderful, naturally white teeth.

WHAT ARE THE CAUSES OF TOOTH DISCOLOURATION?

Most people start off with nice white teeth, but through drinking coffee and tea, certain foods, or perhaps smoking they may become discoloured after a time. Over the years, coloured molecules build up behind the enamel, and because the thickness of enamel decreases with age, the teeth can appear increasingly dark. But now something can be done about that!
Using the Cavex Bite&White ABC Masterkit, you can whiten your teeth quickly, safely and effectively.

HOME WHITENING; AS EASY AS ABC!

The Cavex Bite&White ABC Masterkit is an effective, safe and complete 3-step system which enables whitening to take place in only 30 to 60 minutes a day.

A

ACTIVATING

Cavex Bite&White StainLess

The first step in the process is to clean your teeth thoroughly. Thanks to the intensive action of this toothpaste, superficial stains are removed and the pH value in your mouth is raised temporarily, which helps the whitening gel to take effect more quickly. So your whitening treatment starts by brushing with StainLess for 2 minutes.

B

BLEACHING

Cavex Bite&White gel

This gel is formulated in such a way that it spreads evenly across your teeth so the hydrogen peroxide can start whitening immediately. Because you brushed with the StainLess activator prior, you only need to wear the whitening tray for between 30 and 60 minutes.

Cavex Bite&White ABC Masterkit |
A complete 3-step whitening system
in a single kit.



C

CONDITIONING

Cavex Bite&White ExSense

This soothing gel remineralises your teeth, making them harder as well as minimising the risk of sensitivity. Simply put the gel into your whitening tray and wear for 10 minutes.

MORE INFORMATION

If you would like to know more about whitening your teeth, just ask your dentist; they will be able to tell you everything you need to know about Cavex Bite&White and will guide you through your whitening treatment. Scan the QR code to view the ABC Masterkit videos.



Cavex Bite&White ABC Masterkit |
The complete 3-step solution for home whitening.

CAVEX BITE&WHITE: RECOMMENDED BY DENTISTS ALL OVER THE WORLD

Cavex Bite&White is produced by Cavex Holland. Cavex Holland has been developing and supplying premium products to dentists all over the world for more than 100 years. All Cavex products are extensively tested and comply with guidelines and legislation worldwide. Dentists all over the world recommend our products wholeheartedly.