

CAVEX ALGINATE MIXER 3 PROTOCOL



1 Put the right dosage of alginate and water in the mixing cup.




2 Stir shortly to make cleaning the mixing cup easier and preserve the mixer.

Recommended dosage upper jaw



Recommended dosage lower jaw

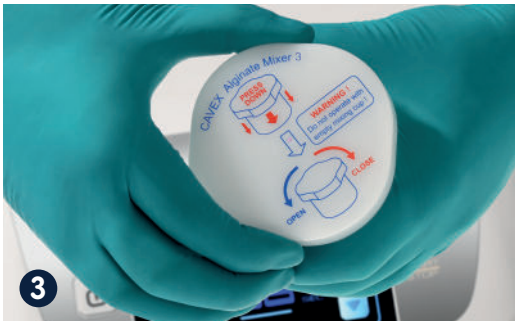

3 Make sure the mixing cup is closed properly by turning the lid clockwise.



4 Place the mixing cup in the mixer and close the lid.

Accessories



5 Select program, press start and wait for the program to finish.



6 Remove the alginate mix from the mixing cup in a single movement and apply it to the tray.



Maintaining and cleaning the Cavex Alginate Mixer 3:

Prevent contamination of the mixer by cleaning it regularly. When dirt comes loose, it can get into the motor and damage the mixer.

1. Dissolve some Cavex GreenClean in a bowl of water.
2. Moisten a cloth (not too wet!) with the GreenClean solution and clean the mixer.
3. Use the stirrer stick with the moist cloth to clean hard-to-reach places.

**YOUR
IMPRESSION
IS OUR
SPECIALTY**