

# CAVEX ALGINATE MIXER II PROTOCOL



1 Measure out the amount of alginate powder and water and place in the mixing cup.



2 Always stir first! This makes cleaning easier and helps keep the appliance in proper working order.



3 Put on the lid and place the cup in the Cavex Alginate Mixer.



4 Close the Cavex Alginate Mixer and press start.



5 Remove the alginate from the cup in a single movement and apply it to the tray.

Recommended dosage upper jaw



Recommended dosage lower jaw



### Accessories



### Maintaining and cleaning the Cavex Alginate Mixer II:

Prevent contamination of the mixer by cleaning it regularly. When dirt comes loose, it can get into the motor and damage the mixer.

1. Dissolve some Cavex GreenClean in a bowl of water.
2. Moisten a cloth (not too wet!) with the GreenClean solution and clean the mixer.
3. Use the stirrer stick with the moist cloth to clean hard- to-reach places.

YOUR  
IMPRESSION  
IS OUR  
SPECIALTY