

PREMIUM RESTORATIVES

QUADRANT FINISENSE

-
For pain caused by exposed dentine. Prevention and relief of post-operative pain after crown and inlay preparation.

- effective within 30 seconds
- works longer than 12 months
- leaves no film/residue on surface
- easy to use
- 150 applications per bottle




CAVEX

QUADRANT FINISENSE


HOW DOES IT WORK?

Finisense can be used for exposed dentine and crown and inlay preparation. Immediate intradentinal sealing halts fluid movement in the tubuli, effectively preventing and relieving post-operative pain.

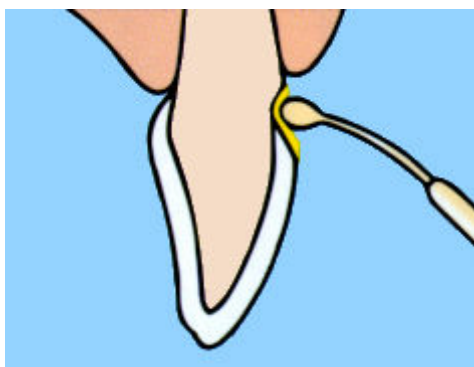

Exposed dentine

- Clean the dentine with for instance pounce
- Carefully dry the dentine surface
- Apply Quadrant FiniSense and let it act on surface for 30 sec.
- Blow dry gently
- Rinse with water thoroughly

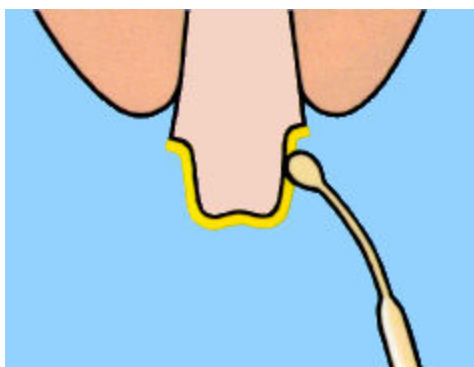

Crown & inlay preparation

- Carefully dry the dentine surface
- Apply Quadrant FiniSense and let it act on surface for 30 sec.
- Blow dry gently
- Rinse with water thoroughly