

PREMIUM ALGINATE SYSTEM

CAVEX ALGINATE ADHESIVE

Improving impressions, reducing deformations

- Improves both mechanical and chemical connection between impression tray and alginate
- Better impression quality due to reduced distortion when the impression tray is removed from the mouth
- Suitable for metal and plastic impression trays
- Very easy to apply with the brush
- No special ventilation measures required
- Minimum of 50 applications per bottle
- Residue can be easily removed with isopropyl alcohol or ethanol (95%)



YOUR
IMPRESSION
IS OUR
SPECIALTY


CAVEX

PREMIUM ALGINATE SYSTEM

- CAVEX ALGINATE ADHESIVE

Instructions:

1. Apply a thin layer of Cavex Alginate Adhesive on the impression tray. Wait 3 to 5 minutes.
2. Apply the alginate impression material.
3. Take the impression as usual.
4. Remove the alginate using Cavex GreenClean.
5. Clean the tray with wisopropyl alcohol or ethanol (95%).

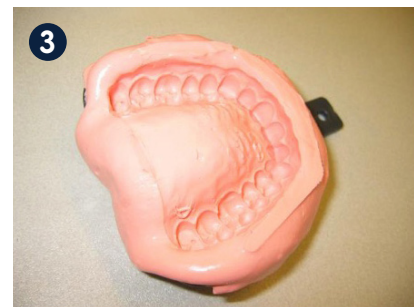

YOUR IMPRESSION IS OUR SPECIALTY

Fustweg 5 | 2031 CJ Haarlem | Nederland | cavex.nl

CAVEX

ZYA150E.01