

PREMIUM WHITENING

CAVEX BITE & WHITE ABC MASTERKIT

-
ACTIVATE - BLEACH - CONDITION

Safe, effective and painless
whitening treatments under
your supervision.



YOUR
IMPRESSION
IS OUR
SPECIALTY

CAVEX

PREMIUM WHITENING

CAVEX BITE & WHITE ABC MASTERKIT

Is your patient looking for a long term and thorough whitening result with minimal sensitivity? Then choose the ABC Masterkit: the most effective and complete whitening treatment.

How does it work?

A

ACTIVATE with Cavex Bite&White StainLess (2-3 minutes).

The teeth should be brushed with the paste to clean the tooth surface thoroughly, so that the whitening gel comes into direct contact with the tooth enamel. In addition, StainLess increases the pH value in the mouth, making the whitening treatment even more effective.

B

BLEACHING with Cavex Bite&White gel (30-60 minutes).

The whitening gel worn in a tray, contains 16% carbamide peroxide which equals 6% hydrogen peroxide. The added sodium fluoride strengthens the enamel and potassium nitrate minimizes any sensitivity.



C

CONDITION with Cavex Bite&White ExSense (5-10 minutes).

The gel can be worn in the same tray and contains a unique composition of hydroxyapatite and a hydro-dispersing sol. This seals the tubules and microcracks in the enamel, minimizing sensitivity. In addition, ExSense contributes to the remineralization and restores the microhardness of the elements.

More information
Scan the QR code
for the video with
more information.



ACTIVATE - BLEACH - CONDITION

YOUR IMPRESSION IS OUR SPECIALTY

CAVEX

Fustweg 5 | 2031 CJ Haarlem | The Netherlands | cavex.nl

ZYBWO9001E 01