—
PREMIUM DENTAL PRODUCTS

CAVEX BITE & WHITE IN-OFFICE

IN-OFFICE

INSTRUCTIONS

—
Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
The Netherlands
+31 23 530 77 00
dental@cavex.nl
cavex.nl

ZB716.01
CE

CAVEX

CAVEX

CAVEX BITE&WHITE IN-OFFICE

IN-OFFICE

INSTRUCTIONS

CAVEX BITE&WHITE IN-OFFICE

Professional whitening system

Cavex Bite&White In-Office gel is based on active oxygen which is added as stabilized, encapsulated hydrogen peroxide. On contact with saliva, the hydrogen peroxide breaks down rapidly into oxygen and water. During this process, free radicals arise which attack the double carbon bonds that are the source of most discoloring (stains) of the teeth.

INDICATIONS FOR USE

Cavex Bite&White is a tooth whitening gel for use in lightening the colour of natural teeth.

CONTRAINDICATIONS

It is recommended to consult a doctor for:

- Pregnant and lactating women.
- Those currently treated for a serious illness.
- Persons under the age of 18 years.

CLINICAL PROCEDURE

- Pre-treatment examen the current dental health and make realistic expectations about final results.
- During the treatment composite or polycarbonate materials remain unaffected. Therefore composite restorations may require replacement after treatment, since only natural teeth will be affected.
- For aesthetic restorations, wait two weeks after the last treatment to allow color stabilization.
- All restorations should be intact or well sealed with adequate temporary materials.
- It is recommended to use a desensitizer (i.e. Cavex ExSense) two weeks prior to, and two weeks after the treatment.
- Determine the initial tooth shade and polish the teeth with a prophyl angle and prophyl paste to remove plaque.

BARRIER PROCEDURE

- Insert a cheek retractor and dry the gingiva with air.
- Apply the barrier to the gingival tissue along the margins of the tooth.
- Cure for 20 sec. with a curing light.

WHITENING PROCEDURE

- Apply a generous layer of Cavex Bite&White In-Office gel onto the teeth. Please use the dosing lines on the pen for reference.
 - One pen can be used for three treatments.
 - After ten to fifteen minutes remove the gel by suction.
 - Repeat the application and removal steps two to three times.
 - Rinse the teeth with water and suction.
- Attention: do not rinse with water during the treatments.

POST-TREATMENT PROCEDURE

- Determine the "After tooth shade".
- Advise the patient to use a sensitivity relief agent (i.e. Cavex ExSense) for at least two week after treatment.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to:

Sensitivity

Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

Gingival irritation

Gel that comes into contact with the gingiva may cause severe soft tissue irritation.

FIRST AID INSTRUCTIONS

- If gel is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and rinse the eye continuously with running water for at least fifteen minutes and contact a doctor.
- If gel gets into contact with clothing, skin or hair, remove contaminated clothing and rinse skin or hair with running water.

STORAGE ADVISE

- Store refrigerated when not in use.
- Short transport (<1 month) is possible under standard conditions.
- Keep away from heat sources and direct sunlight.

DISPOSAL

Dispose of the product in accordance with local waste regulations.

Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty. This also applies where proprietary rights of third parties are ined. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of our advice, any damage occur our liability is limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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