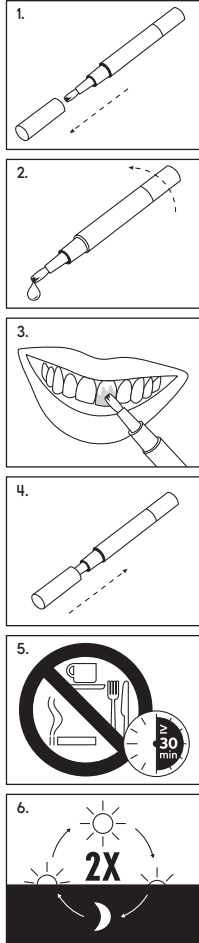
CAVEX BITE&WHITE TOOTH WHITENING PEN

■ Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
The Netherlands
+31 23 530 77 00
dental@ cavex.nl
cavex.nl

ZB708.03

CAVEX

EN (English)

Cavex Bite&White Tooth Whitening Pen

CONTENT

1 x Cavex Tooth Whitening Pen 2 ml

ACTIVE INGREDIENT

± 6 % Hydrogen peroxide

COMPOSITION

PEG-200, water, hydrogen peroxide, poloxamer 124, 1-butyl alcohol, etidronic acid, mentha arvensis extract, EDTA, sodium stannate.

INDICATIONS FOR USE

Cavex Bite&White Tooth Whitening Pen is a tooth whitening gel for use in lightening the colour of natural teeth. Cavex Bite&White Tooth Whitening Pen should only be dispensed by or on the order of a dental professional. Before any whitening procedure, a consultation with a qualified dentist is recommended to ensure underlying oral or health issues.

CONTRAINDICATIONS

It is recommended to consult a doctor, in case of:

- Pregnant and lactating women
- Those currently treated for a serious illness
- Children under the age of 18 years.

INSTRUCTIONS FOR USE (numbers see images):

1. Remove the cap.
2. Turn the pen button until a small droplet of gel appears on the brush.
3. Apply a thick layer of gel directly onto the front surfaces of the teeth.
4. Replace the cap after application.
5. Do not eat, drink or smoke tobacco for at least 30 minutes after each treatment.
6. Apply the gel twice a day.

WARNINGS

- Contains hydrogen peroxide.
 - Avoid contact with eyes.
 - Rinse immediately if product comes into contact with them.
 - Keep out of reach of children.
 - Avoid exposure of known sensitive teeth.
 - Harmful when swallowed.
- Hydrogen peroxide < 6%; is not to be used for persons under 18 years of age; To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

FIRST AID INSTRUCTIONS

- If a large quantity is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and rinse the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to:

Sensitivity

Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

Gingival irritation

Excessive use of whitening gel on the teeth can cause mild to severe tissue irritation. Check if the proper amount of whitening gel is applied and in the correct way. The patient may need to reduce the amount of gel used per tooth.

STORAGE ADVICE

Store at room temperature, max. 25°C. Store in a dry place. Avoid exposure to direct sunlight.

DISPOSAL

Dispose of device in accordance with local waste regulations.

Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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